

Product information

FD 360 cleaning and care of vinyl upholstery



FD 360 at a glance

- Cleaning fluid for deep cleaning and care of synthetic leather surfaces.
- Intensive cleaning action – removes stains of all kinds, spots and discolourations.
- Excellent material compatibility - suitable for synthetic leather coverings.
- Environmentally safe, propellant-free spray bottle.
- Set with 250-mL spray bottle, 5 special sponges and 1 special cloth.

Properties

FD 360 from System Hygiene from Dürr Dental cleans and preserves all synthetic leather upholstery, significantly maintaining the value of treatment chairs and other furnishings with synthetic leather coverings. Simultaneous use of FD 360 and the special cloth cleans and cares for synthetic leather surfaces. The special sponge also removes stubborn contamination, spots, or discolourations, e.g., from non-fixed dyes in clothes, quickly and at pore level. FD 360 provides for deep and intensive, residue-free cleaning and care with a silky shine.

Product composition

The FD 360 mechanism of action is based on a combination of special surfactants, silicone compounds, and avocado oil-based care components.

Directions for use

Apply FD 360 cleaning fluid undiluted. Spray FD 360 on the surface to be cleaned and immediately wipe with the special cloth or scrub with the special sponge for intensive cleaning with FD 360. Using the sponge enables more thorough, deeper cleaning. Combining FD 360 with the special sponge removes even stubborn stains and discolourations from upholstery. Then remove any excess FD 360 by wiping the surface with a dry cloth. Use FD 360 daily, as needed.

Environmental impact

The packaging is made of polyethylene and polypropylene and can thus be recycled or incinerated. For recycling, flush bottles with water.

For further information see safety data sheet.

Shelf life

Do not use the product after the expiry date printed on the packaging.

Package size

Set with 250-mL spray bottle, 5 special sponges and 1 special cloth in the box.

Storage

Store in a well-ventilated place. Keep container tightly closed.

General notes

FD 360 is safe for use on all synthetic leather surfaces. In the case of other surfaces, such as leather, test for material compatibility in an inconspicuous place. Observe the manufacturer's information for the treatment unit. Do not use the special sponges

Product information

FD 360 cleaning and care of vinyl upholstery

on leather. Do not mix FD 360 with other products. Changes in product colour/odour may occur, especially when stored in sunlight. However, these changes in colour and/or odour have no impact on the cleaning efficacy of the product. Use exclusively by qualified personnel. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

Hazard warnings

FD 360 is classified in accordance with the CLP Regulation and is not required to be labelled. For further information see safety data sheet.

pH: 3.5–4.5



Sales

Dürr Dental SE
Höpfigheimer Str. 17
74321 Bietigheim-Bissingen
Germany
www.duerrdental.com
info@duerrdental.com

Manufacturer

orochemie GmbH + Co. KG
Max-Planck-Straße 27
70806 Kornwestheim
Germany
www.orochemie.de
info@orochemie.com