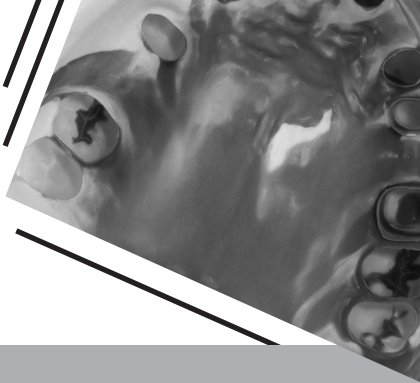
DETAX

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VPS precision impression silicone for dental prosthetics

INDICATION

Detaseal® hydroflow putty 1:1, -soft² putty 1:1, -putty 5:1 – tray material for correction impressions (two-step), for the sandwich and double-mix technique (one-step), and for anatomic impressions.

Detaseal® hydroflow heavy 5:1 – tray material for the double-mix and sandwich technique and anatomic impressions (single-step).

Detaseal® hydroflow lite reg/fast 1:1, Xlite reg/fast 1:1 – correction material, sprayable, for correction impressions (two-step), for the double-mix and sandwich technique, and for relining impressions (single-step).

Detaseal hydroflow mono® 1:1, mono® 5:1 – tray material, for single-phase impressions, for the double-mix and sandwich technique, for anatomic impressions, over- and fixation impressions, relining impressions and transfer impressions.

monoprint supra 1:1 – for transfer impressions in implantology, transfer impressions with closed and open trays, direct impressions, over- and fixation impressions in the combination technique and single-phase impressions.

Detaseal® antilock 1:1 – Block-out silicone for undercut areas of fixed prosthetic and orthodontic restorations and for periodontally damaged, loosened teeth and greatly widened interdental areas.

Detaseal® function 1:1 – Special impression material for functional impressions with extended curing phase and functional margin design in total prosthetics.

PATIENT TARGET GROUP

Persons being treated in the context of a dental procedure.

INTENDED USERS

Dentist

PRODUCT DESCRIPTION

The vinyl polysiloxanes (VPS) of the Detaseal® hydroflow series and monoprint supra are addition-curing elastomeric impression materials.

Detaseal® heavy, mono, lite, Xlite and monoprint supra form hydrophilic surfaces.

Detaseal® silicones have differing consistencies according to DIN EN ISO 4823 type 0 – 3 (see table page 16).

METERING AND MIXING IS CARRIED OUT USING DIFFERENT SYSTEMS

Cans 1:1: putty, putty soft² – manual metering and kneading

Cartridges 5:1: putty, heavy, mono – common dynamic mixers

Cartridges 1:1: lite, Xlite, mono, monoprint supra, antilock – Automix2 cartridge system

PROCEDURE

TRAY, ADHESIVE

Select a suitable impression tray according to the impression technique. Apply trayloc® A to optimize silicone adhesion.

MANUAL METERING AND MIXING

putty – The metering is carried out at a mixing ratio of 1:1 with the enclosed metering spoons.

Detaseal® function – dispense from the dosing syringes. Maintain the 1:1 dosing by using equal strand lengths of base and catalyst. Mixing is carried out manually by kneading until a homogeneous mixture or uniform color, respectively, is reached. Make sure that only base and catalyst components from the same batch number are used. Immediately close again after use. Do not swap lids, closures or metering spoons.

DYNAMIC MIXERS 5:1 – CARTRIDGES PUTTY, HEAVY, MONO

Jumbopack – Unlock and pull off the tamper-proof closure on the cartridge by lifting the locking lever (Fig. 1). Insert the cartridge into the metering unit. Start the device and spread the material until it is being evenly emitted. Discard the first inhomogeneous mass, then attach the dynamic mixer (yellow). Fixate the mixer by attaching and turning the yellow rotating ring to the right (Fig. 2).

Jumbo plus – Before first use, turn the blue plug to the left and pull it out (Fig. 3). Check the outlet openings. Then screw on the dynamic mixer (blue) (Fig. 4). Make sure that the positioning aid faces upward. Discard the first material that comes out, approx. 3 cm. Do not fill the tray before the mixture is homogeneous. Leave the filled mixer as a closure on the cartridge until the next application. Before the next application, remove the used mixer and check the outlet openings of the cartridge for clogging; then attach a new mixer. In order to ensure reliable application, we recommend using exclusively the matching dynamic mixers by DETAX. Please read and understand the notes by the manufacturer of the mixer.

AUTOMIX2 CARTRIDGES 1:1 – LITE, XLITE, MONO, MONOPRINT SUPRA, ANTILOCK

The cartridges are emptied with the Automix2 system pistol (Fig. 5). After inserting the cartridge into the mixing gun, remove the cartridge closure by turning it. Squeeze out a small amount of material until silicone is being uniformly emitted from both outlet openings (Fig. 6). Attach the mixing cannula in accordance with the guides on the cannula and cartridge, and lock it by turning it in the opposite direction (Fig. 7). Emit material with even pressure. Spread a small amount of material before use and visually check that the base and catalyst pastes are homogeneously mixed (Fig. 8). Only then can the metering be carried out individually. After use, leave the mixing cannula on the cartridge until the next application.

DISINFECTION

After taking the impression out of the mouth, rinse it under running lukewarm water. Subsequent disinfection can be carried out with suitable commercially available disinfecting solutions according to the manufacturer's instructions.

PRODUCING THE LABORATORY MODEL

With dental plaster type III – V, this can be done already about 30 minutes after removal from the mouth.

TRAY CLEANING

Remove cured material mechanically. Placing in acetone or isopropanol dissolves trayloc® A. Use only in well-ventilated areas. Then clean and disinfect the tray as usual.

SAFETY INFORMATION

- ▶ Only for the specified intended use by trained personnel.
- ▶ monoprint supra: Reliable fixation of transfer posts and retention parts is significantly improved by application of trayloc® A.
- ▶ Detaseal® antilock: Does not react with polyether impression materials, condensation-curing silicones and alginates; pay attention to sufficient material thickness.
- ▶ In order to prevent locking in marked undercuts, these can be filled up with Detaseal® antilock in advance to facilitate removal of single-step impressions from the mouth. Do not apply Detaseal® antilock in preparation areas. Use only as block-out material.
- ▶ Detaseal® antilock does not react with polyether impression materials, condensation-curing silicones and alginates.
- ▶ Solutions used prior to impression taking (e.g. retraction solutions) may affect the curing reaction and must therefore be removed completely.

- ▶ Latex gloves and latex-contaminated surfaces can affect curing.
- ▶ In case of stiffness or clogging discard the cartridge; do not use force for application!
- ▶ Do not leave any residues of the impression material in the patient's mouth or sulcus, and make sure to avoid inadvertent pressing of material into areas outside the oral cavity through mouth-antrum connections or malformations of the palate.
- ▶ Cured silicon materials are chemically resistant – avoid stains on clothing.
- ▶ Wear suitable personal protective equipment during the finishing of the cured material.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

NOTES

- ▶ DETAX shall not be held liable for any damage caused by misuse.
- ▶ Read and understand the safety data sheet!

For users and/or patients:

Any serious incidents occurring in relation to this product should be reported immediately to incident@detax.de and to the competent authority of the Member State in which the user and/or patient is established.

CONTRAINDICATION

Do not use this material in case of known allergies to any of the ingredients, or contact allergies. Silicone impression materials have been tried and tested millions of times; undesired effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

DISPOSAL

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

Technical Specifications

	DIN EN ISO 4823 TYPE	Mixing volume	Dosing	Product color base/catalyst
Detaseal® hydroflow putty	type 0, putty	2 × 250 ml can	1:1	green / white
Detaseal® hydroflow soft ² putty	type 0, putty	2 × 500 ml can	1:1	yellow / white
Detaseal® hydroflow putty Jumbopack	type 0, putty	380-ml cartridge	5:1	green / white
Detaseal® hydroflow putty Jumbo Plus	type 0, putty	380-ml cartridge	5:1	green / white
Detaseal® hydroflow heavy Jumbo	type 1, heavy-bodied	380-ml cartridge	5:1	green / white
Detaseal® hydroflow heavy Jumbo Plus	type 1, heavy-bodied	380-ml cartridge	5:1	green / white
Detaseal® hydroflow mono	type 2, medium-bodied	50-ml cartridge	1:1	yellow / white
Detaseal® hydroflow mono Jumbopack	type 2, medium-bodied	380-ml cartridge	5:1	yellow / white
Detaseal® hydroflow mono Jumbo Plus	type 2, medium-bodied	380-ml cartridge	5:1	yellow / white
Detaseal® hydroflow lite regular set	type 3, light-bodied	50-ml cartridge	1:1	pink / white
Detaseal® hydroflow lite fast set	type 3, light-bodied	50-ml cartridge	1:1	pink / white
Detaseal® hydroflow Xlite regular set	type 3, light-bodied	50-ml cartridge	1:1	blue / white
Detaseal® hydroflow Xlite fast set	type 3, light-bodied	50-ml cartridge	1:1	blue / white
monoprint supra	type 2, medium-bodied	50-ml cartridge	1:1	yellow / white
Detaseal® function	type 2, medium-bodied	2 × 80-ml dispenser	1:1	red / yellow
Detaseal® antilock	–	50-ml cartridge	1:1	pink / white

Mixing time sec. approx.	Open time* min. approx.	Time in mouth min. approx.	Curing time* min. approx.	Mixer
30	1:30	2:00	3:30	n/a
30	1:15	1:30	2:45	n/a
n/a	1:30	2:30	4:00	dynamic mixer yellow
n/a	1:30	2:30	4:00	dynamic mixer blue
n/a	1:30	2:00	3:30	dynamic mixer yellow
n/a	1:30	2:00	3:30	dynamic mixer blue
n/a	1:30	2:00	3:30	mixing cannulas pink, T-Mixer pink
n/a	1:30	2:00	3:30	dynamic mixer yellow
n/a	1:30	2:00	3:30	dynamic mixer blue
n/a	2:00	2:00	4:00	mixing cannulas yellow, T-Mixer yellow
n/a	1:00	1:30	2:30	mixing cannulas yellow, T-Mixer yellow
n/a	2:00	2:00	4:00	mixing cannulas yellow, T-Mixer yellow
n/a	1:00	1:30	2:30	mixing cannulas yellow, T-Mixer yellow
n/a	2:00	2:00	4:00	mixing cannulas green, T-mixer green
30	2:30	3:30	6:00	n/a
n/a	1:20	1:25	2:45	mixing cannulas yellow, T-Mixer yellow

Symbols

MD Medical Device

LOT see bottom

Processing:
At 23 °C ± 2 °C,
50 ± 5%
rel. humidity

Storage:

15 °C
59 °F

25 °C
77 °F

* from start of
mixing at
23 °C ± 2 °C,
50 ± 5 % rel.
humidity. Higher
temperatures
shorten, lower
temperatures
extend the indi-
cated times.

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Ved 23 °C ± 2 °C, 50 ± 5 % rel. luftfugtighed

Tekniske data

	DIN EN ISO 4823 TYPE	Blandingsvolumen	Dosering	Produktfarve base/ katalysator
Detaseal® hydroflow putty	Type 0, putty	2 x 250 ml dosis	1:1	grøn / hvid
Detaseal® hydroflow soft ² putty	Type 0, putty	2 x 500 ml dosis	1:1	gul / hvid
Detaseal® hydroflow putty Jumbopack	Type 0, putty	380 ml patron	5:1	grøn / hvid
Detaseal® hydroflow putty Jumbo Plus	Type 0, putty	380 ml patron	5:1	grøn / hvid
Detaseal® hydroflow heavy Jumbo	Type 1, heavy-bodied	380 ml patron	5:1	grøn / hvid
Detaseal® hydroflow heavy Jumbo Plus	Type 1, heavy-bodied	380 ml patron	5:1	grøn / hvid
Detaseal® hydroflow mono	Type 2, medium-bodied	50 ml patron	1:1	gul / hvid
Detaseal® hydroflow mono Jumbopack	Type 2, medium-bodied	380 ml patron	5:1	gul / hvid
Detaseal® hydroflow mono Jumbo Plus	Type 2, medium-bodied	380 ml patron	5:1	gul / hvid
Detaseal® hydroflow lite regular set	Type 3, light-bodied	50 ml patron	1:1	pink / hvid
Detaseal® hydroflow lite fast set	Type 3, light-bodied	50 ml patron	1:1	pink / hvid
Detaseal® hydroflow Xlite regular set	Type 3, light-bodied	50 ml patron	1:1	blå / hvid
Detaseal® hydroflow Xlite fast set	Type 3, light-bodied	50 ml patron	1:1	blå / hvid
monoprint supra	Type 2, medium-bodied	50 ml patron	1:1	gul / hvid
Detaseal® function	Type 2, medium-bodied	2 x 80 ml dispenser	1:1	rød / gul
Detaseal® antilock	–	50 ml patron	1:1	pink / hvid

Blandingstid sek. ca.	Forarbejdningstid* min. ca.	Opbevaringstid i munden min. ca.	Hærdetid* min. ca.	Blander
30	1:30	2:00	3:30	bortfalder
30	1:15	1:30	2:45	bortfalder
bortfalder	1:30	2:30	4:00	dynamisk blander gul
bortfalder	1:30	2:30	4:00	dynamisk blander blå
bortfalder	1:30	2:00	3:30	dynamisk blander gul
bortfalder	1:30	2:00	3:30	dynamisk blander blå
bortfalder	1:30	2:00	3:30	Blandingskanyler rosa T-mixer rosa
bortfalder	1:30	2:00	3:30	dynamisk blander gul
bortfalder	1:30	2:00	3:30	dynamisk blander blå
bortfalder	2:00	2:00	4:00	Blandingskanyler gul, T-mixer gul
bortfalder	1:00	1:30	2:30	Blandingskanyler gul, T-mixer gul
bortfalder	2:00	2:00	4:00	Blandingskanyler gul, T-mixer gul
bortfalder	1:00	1:30	2:30	Blandingskanyler gul, T-mixer gul
bortfalder	2:00	2:00	4:00	Blandingskanyler grøn, T-mixer grøn
30	2:30	3:30	6:00	bortfalder
bortfalder	1:20	1:25	2:45	Blandingskanyler gul, T-mixer gul

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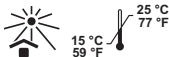
na 23 °C $\pm$ 2 °C, 50 $\pm$ 5% rel. vlažnost zraka

Tehnički podaci

	DIN EN ISO 4823 TYP	Volumen miješanja	Doziranje	Boja proizvoda baza/ katalizator
Detaseal® hydroflow putty	tip 0, putty	2 x 250 ml doza	1:1	zelena / bijeli
Detaseal® hydroflow soft ² putty	tip 0, putty	2 x 500 ml doza	1:1	žuta / bijeli
Detaseal® hydroflow putty Jumbopack	tip 0, putty	380 ml spremnik	5:1	zelena / bijeli
Detaseal® hydroflow putty Jumbo Plus	tip 0, putty	380 ml spremnik	5:1	zelena / bijeli
Detaseal® hydroflow heavy Jumbo	tip 1, heavy-bodied	380 ml spremnik	5:1	zelena / bijeli
Detaseal® hydroflow heavy Jumbo Plus	tip 1, heavy-bodied	380 ml spremnik	5:1	zelena / bijeli
Detaseal® hydroflow mono	tip 2, medium-bodied	50 ml spremnik	1:1	žuta / bijeli
Detaseal® hydroflow mono Jumbopack	tip 2, medium-bodied	380 ml spremnik	5:1	žuta / bijeli
Detaseal® hydroflow mono Jumbo Plus	tip 2, medium-bodied	380 ml spremnik	5:1	žuta / bijeli
Detaseal® hydroflow lite regular set	tip 3, light-bodied	50 ml spremnik	1:1	ružičasta / bijeli
Detaseal® hydroflow lite fast set	tip 3, light-bodied	50 ml spremnik	1:1	ružičasta / bijeli
Detaseal® hydroflow Xlite regular set	tip 3, light-bodied	50 ml spremnik	1:1	plava / bijeli
Detaseal® hydroflow Xlite fast set	tip 3, light-bodied	50 ml spremnik	1:1	plava / bijeli
monoprint supra	tip 2, medium-bodied	50 ml spremnik	1:1	žuta / bijeli
Detaseal® function	tip 2, medium-bodied	2 x 80 ml dozator	1:1	crvena / žuti
Detaseal® antilock	–	50 ml spremnik	1:1	ružičasta / bijeli

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	DIN EN ISO 4823 TYPE	Blandingsvolum	Dosering	Produktfarge base/ katalysator
Detaseal® hydroflow putty	Type 0, putty	2 x 250 ml boks	1:1	grønn/hvit
Detaseal® hydroflow soft ² putty	Type 0, putty	2 x 500 ml boks	1:1	gul/hvit
Detaseal® hydroflow putty Jumbopack	Type 0, putty	380 ml kassett	5:1	grønn/hvit
Detaseal® hydroflow putty Jumbo Plus	Type 0, putty	380 ml kassett	5:1	grønn/hvit
Detaseal® hydroflow heavy Jumbo	Type 1, heavy-bodied	380 ml kassett	5:1	grønn/hvit
Detaseal® hydroflow heavy Jumbo Plus	Type 1, heavy-bodied	380 ml kassett	5:1	grønn/hvit
Detaseal® hydroflow mono	Type 2, medium-bodied	50 ml kassett	1:1	gul/hvit
Detaseal® hydroflow mono Jumbopack	Type 2, medium-bodied	380 ml kassett	5:1	gul/hvit
Detaseal® hydroflow mono Jumbo Plus	Type 2, medium-bodied	380 ml kassett	5:1	gul/hvit
Detaseal® hydroflow lite regular set	Type 3, light-bodied	50 ml kassett	1:1	rosa/hvit
Detaseal® hydroflow lite fast set	Type 3, light-bodied	50 ml kassett	1:1	rosa/hvit
Detaseal® hydroflow Xlite regular set	Type 3, light-bodied	50 ml kassett	1:1	blå/hvit
Detaseal® hydroflow Xlite fast set	Type 3, light-bodied	50 ml kassett	1:1	blå/hvit
monoprint supra	Type 2, medium-bodied	50 ml kassett	1:1	gul/hvit
Detaseal® function	Type 2, medium-bodied	2 x 80 ml dispenser	1:1	rød/gul
Detaseal® antilock	–	50 ml kassett	1:1	rosa/hvit

Blandetid sek. ca.	Bearbejdings- tid* min. ca.	Holdetid i munnen min. ca.	Herdetid* min. ca.	Mikser
30	1:30	2:00	3:30	bortfaller
30	1:15	1:30	2:45	bortfaller
bortfaller	1:30	2:30	4:00	dynamisk mikser gul
bortfaller	1:30	2:30	4:00	dynamisk mikser blå
bortfaller	1:30	2:00	3:30	dynamisk mikser gul
bortfaller	1:30	2:00	3:30	dynamisk mikser blå
bortfaller	1:30	2:00	3:30	Blandekanyler rosa, T-mikser rosa
bortfaller	1:30	2:00	3:30	dynamisk mikser gul
bortfaller	1:30	2:00	3:30	dynamisk mikser blå
bortfaller	2:00	2:00	4:00	Blandekanyler gule, T-mikser gul
bortfaller	1:00	1:30	2:30	Blandekanyler gule, T-mikser gul
bortfaller	2:00	2:00	4:00	Blandekanyler gule, T-mikser gul
bortfaller	1:00	1:30	2:30	Blandekanyler gule, T-mikser gul
bortfaller	2:00	2:00	4:00	Blandekanyler grøn, T-mikser grøn
30	2:30	3:30	6:00	bortfaller
bortfaller	1:20	1:25	2:45	Blandekanyler gule, T-mikser gul

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La 23 °C ± 2 °C, 50 ± 5% umiditate relativă aer

Date tehnice

	DIN EN ISO 4823 TYP	Volum de amestec	Dozare	Culoare produs bază/catalizator
Detaseal® hydroflow putty	Tip 0, putty	2 x 250 ml doză	1:1	verde/alb
Detaseal® hydroflow soft ² putty	Tip 0, putty	2 x 500 ml doză	1:1	galben/alb
Detaseal® hydroflow putty Jumbopack	Tip 0, putty	Cartuș 380 ml	5:1	verde/alb
Detaseal® hydroflow putty Jumbo Plus	Tip 0, putty	Cartuș 380 ml	5:1	verde/alb
Detaseal® hydroflow heavy Jumbo	Tip 1, heavy-bodied	Cartuș 380 ml	5:1	verde/alb
Detaseal® hydroflow heavy Jumbo Plus	Tip 1, heavy-bodied	Cartuș 380 ml	5:1	verde/alb
Detaseal® hydroflow mono	Tip 2, medium-bodied	Cartuș 50 ml	1:1	galben/alb
Detaseal® hydroflow mono Jumbopack	Tip 2, medium-bodied	Cartuș 380 ml	5:1	galben/alb
Detaseal® hydroflow mono Jumbo Plus	Tip 2, medium-bodied	Cartuș 380 ml	5:1	galben/alb
Detaseal® hydroflow lite regular set	Tip 3, light-bodied	Cartuș 50 ml	1:1	roz/alb
Detaseal® hydroflow lite fast set	Tip 3, light-bodied	Cartuș 50 ml	1:1	roz/alb
Detaseal® hydroflow Xlite regular set	Tip 3, light-bodied	Cartuș 50 ml	1:1	albastru/alb
Detaseal® hydroflow Xlite fast set	Tip 3, light-bodied	Cartuș 50 ml	1:1	albastru/alb
monoprint supra	Tip 2, medium-bodied	Cartuș 50 ml	1:1	galben/alb
Detaseal® function	Tip 2, medium-bodied	Dozator 2 x 80 ml	1:1	roșu/galben
Detaseal® antilock	–	Cartuș 50 ml	1:1	roz/alb

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Tekniska data

	DIN EN ISO 4823 TYP	Blandningsvolym	Dosering	Produktfärg bas/katalysator
Detaseal® hydroflow putty	Typ 0, putty	2 x 250 ml dosa	1:1	grön / vit
Detaseal® hydroflow soft ² putty	Typ 0, putty	2 x 500 ml dosa	1:1	gul / vit
Detaseal® hydroflow putty Jumbopack	Typ 0, putty	380 ml patron	5:1	grön / vit
Detaseal® hydroflow putty Jumbo Plus	Typ 0, putty	380 ml patron	5:1	grön / vit
Detaseal® hydroflow heavy Jumbo	Typ 1, heavy-bodied	380 ml patron	5:1	grön / vit
Detaseal® hydroflow heavy Jumbo Plus	Typ 1, heavy-bodied	380 ml patron	5:1	grön / vit
Detaseal® hydroflow mono	Typ 2, medium-bodied	50 ml patron	1:1	gul / vit
Detaseal® hydroflow mono Jumbopack	Typ 2, medium-bodied	380 ml patron	5:1	gul / vit
Detaseal® hydroflow mono Jumbo Plus	Typ 2, medium-bodied	380 ml patron	5:1	gul / vit
Detaseal® hydroflow lite regular set	Typ 3, light-bodied	50 ml patron	1:1	skär / vit
Detaseal® hydroflow lite fast set	Typ 3, light-bodied	50 ml patron	1:1	skär / vit
Detaseal® hydroflow Xlite regular set	Typ 3, light-bodied	50 ml patron	1:1	blå / vit
Detaseal® hydroflow Xlite fast set	Typ 3, light-bodied	50 ml patron	1:1	blå / vit
monoprint supra	Typ 2, medium-bodied	50 ml patron	1:1	gul / vit
Detaseal® function	Typ 2, medium-bodied	2 x 80 ml dispenser	1:1	röd / gul
Detaseal® antilock	–	50 ml patron	1:1	skär / vit

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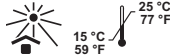
İşleme:

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Teknik veriler

	DIN EN ISO 4823 TİP	Karışım hacmi	Dozaj	Temel/katalizör ürün rengi
Detaseal® hydroflow putty	Tip 0, putty	2 x 250 ml kutu	1:1	yeşil / beyaz
Detaseal® hydroflow soft ² putty	Tip 0, putty	2 x 500 ml kutu	1:1	sarı / beyaz
Detaseal® hydroflow putty Jumbopack	Tip 0, putty	380 ml kartuş	5:1	yeşil / beyaz
Detaseal® hydroflow putty Jumbo Plus	Tip 0, putty	380 ml kartuş	5:1	yeşil / beyaz
Detaseal® hydroflow heavy Jumbo	Tip 1, heavy-bodied	380 ml kartuş	5:1	yeşil / beyaz
Detaseal® hydroflow heavy Jumbo Plus	Tip 1, heavy-bodied	380 ml kartuş	5:1	yeşil / beyaz
Detaseal® hydroflow mono	Tip 2, medium-bodied	50 ml kartuş	1:1	sarı / beyaz
Detaseal® hydroflow mono Jumbopack	Tip 2, medium-bodied	380 ml kartuş	5:1	sarı / beyaz
Detaseal® hydroflow mono Jumbo Plus	Tip 2, medium-bodied	380 ml kartuş	5:1	sarı / beyaz
Detaseal® hydroflow lite regular set	Tip 3, light-bodied	50 ml kartuş	1:1	pembe / beyaz
Detaseal® hydroflow lite fast set	Tip 3, light-bodied	50 ml kartuş	1:1	pembe / beyaz
Detaseal® hydroflow Xlite regular set	Tip 3, light-bodied	50 ml kartuş	1:1	mavi / beyaz
Detaseal® hydroflow Xlite fast set	Tip 3, light-bodied	50 ml kartuş	1:1	mavi / beyaz
monoprint supra	Tip 2, medium-bodied	50 ml kartuş	1:1	sarı / beyaz
Detaseal® function	Tip 2, medium-bodied	2 x 80 ml dağıtıcı	1:1	kırmızı / sarı
Detaseal® antilock	–	50 ml kartuş	1:1	pembe / beyaz

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Detaseal® hydroflow putty

Standard packing 02727

1 jar base each 250 ml
1 jar catalyst each 250 ml
2 measuring scoops

Multi packing 02728

2 jar base each 250 ml
2 jar catalyst each 250 ml
2 measuring scoops

Jumbo packing 03283

1 cartridge base + catalyst 380 ml
10 dynamic mixers, yellow
1 rotating ring, yellow

Double Jumbo packing 03385

2 cartridges base + catalyst each 380 ml
2 dynamic mixers, yellow
1 rotating ring, yellow

Jumbo Plus 02273

1 cartridge base + catalyst each 380 ml
10 dynamic mixers, blue

Detaseal® hydroflow soft² putty

Standard packing 02880

1 jar base each 500 ml
1 jar catalyst each 500 ml
2 measuring scoops

Detaseal® hydroflow heavy

Jumbo packing 02729

1 cartridge base + catalyst 380 ml
10 dynamic mixers, yellow
1 rotating ring, yellow

Double Jumbo packing 03388

2 cartridges base + catalyst each 380 ml
2 dynamic mixers, yellow
1 rotating ring, yellow

Jumbo Plus 02241

1 cartridge base + catalyst 380 ml
10 dynamic mixers, blue

Detaseal® hydroflow mono

Standard packing 02754

2 cartridges each 50 ml
6 mixing cannulas, green

Multi packing 02757

4 cartridges each 50 ml
12 mixing cannulas, green

Jumbo packing 03373

1 cartridge base + catalyst 380 ml
10 dynamic mixers, yellow
1 rotating ring, yellow

Double Jumbo packing 03386

2 cartridges base + catalyst 380 ml
2 dynamic mixers, yellow
1 rotating ring, yellow

Jumbo Plus 02272

1 cartridge base + catalyst 380 ml
10 rotating ring, blue

Detaseal® hydroflow lite

Standard packing 02730

regular set
2 cartridges each 50 ml
12 mixing cannulas, yellow

Multi packing 02732

regular set
4 cartridges each 50 ml
24 mixing cannulas, yellow

Standard packing 02733

fast set
2 cartridges each 50 ml
12 mixing cannulas, yellow

Multi packing 02740

fast set
4 cartridges each 50 ml
24 mixing cannulas, yellow

Detaseal® hydroflow Xlite

Standard packing 02741

regular set
2 cartridges each 50 ml
12 mixing cannulas, yellow

<u>Multi packing</u> regular set 4 cartridges each 50 ml 24 mixing cannulas, yellow	02743	Dynamic mixers yellow 40 pcs.	02833
<u>Standard packing</u> fast set 2 cartridges each 50 ml 12 mixing cannulas, yellow	02744	Rotating ring yellow 1 pc.	02836
<u>Multi packing</u> fast set 4 cartridges each 50 ml 24 mixing cannulas, yellow	02745	Dynamic mixers blue 40 pcs.	02281
		Mixing gun 1:1 / 2:1 1 pcs.	02699
Detaseal® function		Mixing cannulas yellow 48 pcs.	02706
<u>Standard packing</u> 2 dispenser base + catalyst each 80 ml 1 moulding syringe	02521	rose 48 pcs.	02707
		green 48 pcs.	02705
monoprint supra		T-Mixer yellow 25 pcs.	03615
<u>Standard packing</u> 2 cartridges base + catalyst each 50 ml 6 mixing cannulas, green	02525	rose 25 pcs.	03614
		green 25 pcs.	03616
Detaseal® antilock		Intra-Oral-Tips yellow 96 pcs.	02712
<u>Standard packing</u> 1 cartridge each 50 ml 6 mixing cannulas, yellow 6 Intra-Oral-Tips	03535	white 96 pcs.	02345
		trayloc® A bottle with brush, 17 ml vaporizer of, 15 ml	03098 03392

This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and extend across the width of the page. There are no margins, text, or other markings on the paper.

This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

DETAX

GmbH & Co. KG

Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany

Telefon: 07243/510-0 · Fax: 07243/510-100

www.detax.de · post@detax.de

Made in
Germany

02/2021
3.1

Caution: Federal U.S. law restricts this device to sale by or on the order of a dentist (or trained specialist personnel).